

Instructions for Trimming the CyMedica QB1® and e-vive™ Knee Brace Straps

Steps to Trim the Straps:

1. **Fit the patient with the brace** and determine appropriate strap length.




Note: The post-op brace will be provided pre-attached to the garment inside the package. The images and instructions below are demonstrated without the garment/brace for better visibility of the straps. **The straps do not need to be removed from the brace for trimming.**


The post-op brace Alligator tabs allow for easy strap trimming:




2. **Remove Alligator tab** from strap ends by pulling up on the fabric adjacent to the rubber logo on both sides of the strap.

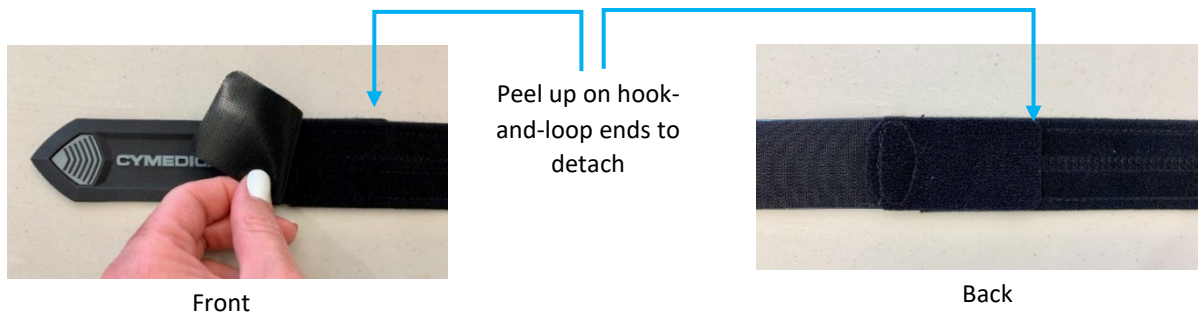

3. **Trim excess material** starting from the end of the strap. Take care to not cut too short.

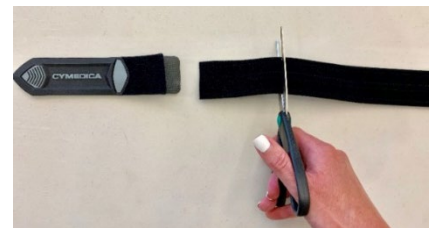

4. **Replace Alligator tab** by re-attaching the fabric onto both ends of the strap with the hook-and-loop material facing down.

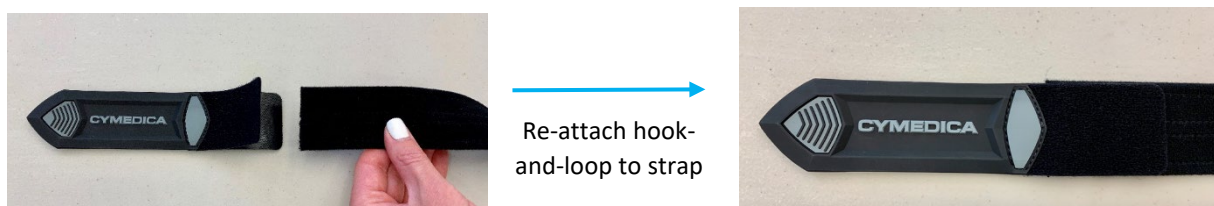

5. **Secure all straps** of the post-op brace. The brace should fit snug to the leg.